

Disposable Respirator Fitting Instructions

1. Users must follow instructions each time respirator is worn.

2. It's the wearer's responsibility to obtain a proper fit.

3. To choose the correct model/size Moldex Respirator you must fit test and user seal check.


4. If you cannot obtain a proper fit do not enter the contaminated area.

5. OSHA regulation 29 CFR 1910.134 (f) requires that the user must be fit tested before being assigned a respirator and annually thereafter.
6. Use an appropriate OSHA (or other official regulatory agency) approved fit test method.

7. User must be clean-shaven. Any facial hair, such as beards or long sideburns, may prevent the respirator from fitting properly.

8. Re-fit test when there are any changes to the wearers physical condition that may affect fit and/or when changing from any model/size to another respirator.

2-STRAP DISPOSABLE RESPIRATORS

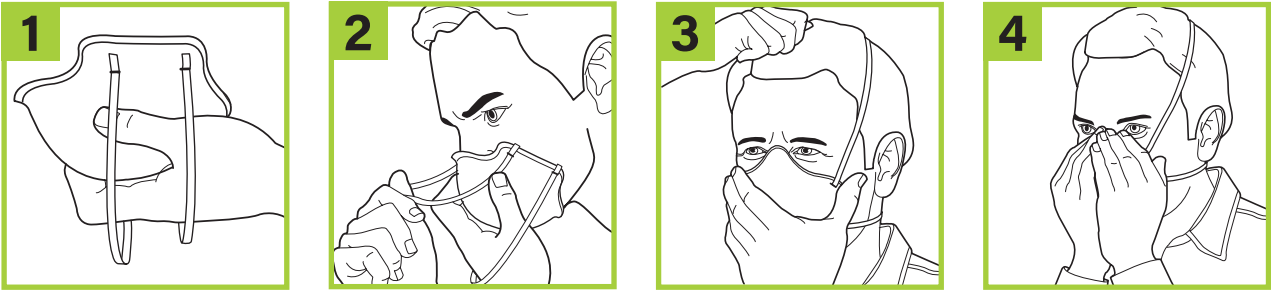

1. Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand (Fig 1).

2. Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must not be creased inside respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to around neck. **Do not wear with only one strap because it may affect fit.** (Fig 2).
3. Adjust respirator for comfortable fit (Fig 3).

4. Each time user enters contaminated area seal should be fit checked. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death (Fig 4).

SMARTSTRAP® RESPIRATORS



1. Pull adjustment clip and strap fully below the bottom of the mask (Fig 1).


2. Place strap around neck so outside of shell is against chest (Fig 2).

3. a) Fit mask to face and pull top of strap to crown of head. Nose cushion or foam cushion must be uncreased inside of respirator. b) Always use both top and bottom straps (Fig 3).

4. To tighten pull both strap tabs (Fig 4). To loosen, grasp both sides of neck strap and pull.
5. Each time user enters contaminated area respirator must be seal checked. Cover front of respirator by cupping both hands. (Fig 5). **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, tighten or adjust strap by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.

6. To hang around neck, pull top strap away from head, then pull mask down (Fig 2).

HANDYSTRAP® RESPIRATORS

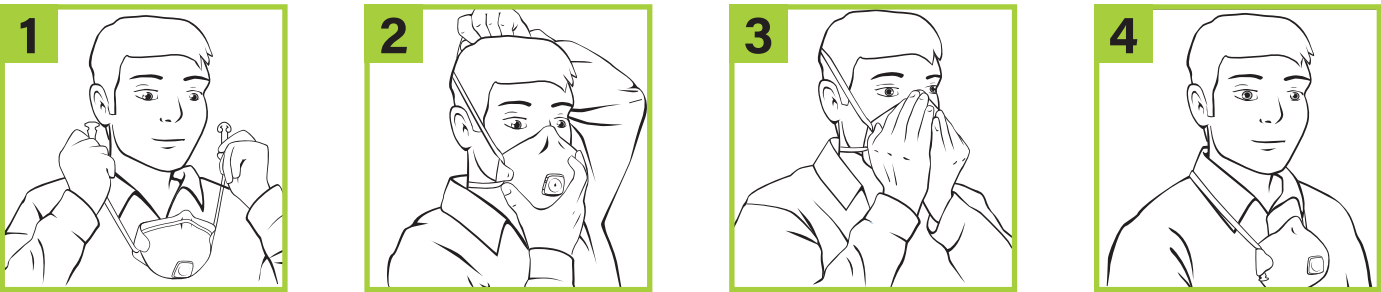

1. Attach buckle behind neck with shell against chest.

2. a) Fit mask to face and pull top of strap to crown of head. Nose cushion must be uncreased inside respirator. b) Always use top and bottom part of strap. Adjust tension for a comfortable fit. When changing from any model/size to another respirator, you must fit test.

3. Each time user enters contaminated area respirator must be seal checked. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust strap by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.
4. To hang around neck:
a) Undo buckle.
b) Allow strap to fall around neck.
c) Pull mask down to chest.
OR
a) Pull top strap forward away from head.
b) Adjust tension to allow mask to hang freely.

MOLDEX®

TECHNICAL HELP LINE
For technical assistance call Moldex Technical Service Department
at 1 (310) 837-6500, ext. 512 or 1 (800) 421-0668, ext. 512 or email tech@moldex.com.

U.S. & CANADA
10111 JEFFERSON BOULEVARD
CULVER CITY, CA 90232
TEL: +1 (310) 837-6500 EXT. 550
FAX: +1 (310) 837-9563
sales@moldex.com
www.moldex.com

BRAZIL
AVENIDA IPIRANGA, 104-22
ANDAR, SÃO PAULO,
SP CEP:01046-010
BRASIL
FONE/FAX: 113255-0555
CEL: +55-11-99486-4083
moldexbrazil@moldex.com
moldex.com/pt-br

AUSTRALIA
P.O. BOX 4086
CARLINGFORD NORTH, NSW 2118
TEL: 1300 MOLDEX (1 300 665 339)
moldexaustralia@moldex.com
moldex.com/au

NEW ZEALAND
3 SPENCE ROAD
HENDERSON, AUCKLAND, NZ
TEL: +64 027 243 1244
moldexnz@moldex.com
moldex.com/nz

MOLDEX JAPAN K.K.
FIS BLDG, 403
SHIMOMARUYA-CHO,
NAKAGYO-KU, KYOTO, 604-8006
JAPAN
TEL: 011-81757785070
FAX: 011-81752568657
moldexjapan@moldex.com

ATTENTION: If these respirators are used against any contaminants it must be used in accordance with the appropriate local, state and federal regulations including 29 CFR 1910.134 as a minimum.

Moldex® and the PVC-Free logo are registered trademarks of Moldex.



9900-037 REV B 11/24